viraShade®

Danish Design. Design registered.
© mencke & vagnby.

Made in Hong Kong SAR by
International Design House Ltd.

CE 2777 (EU) 2016/425


4 897032 181854


VISOR

US/GB viraShade® is a flat-packed face shield, designed to protect against direct splashes or droplets that may contain viruses, bacteria, blood, other body fluids or certain chemicals. It is assembled in seconds, by pulling the elastic headband outwards. Hereby the m-frame is formed, giving the shield its outstanding structural strength, while it is still flexible, and will adjust to any head size and shape, ensuring a perfect fit for all. This makes the viraShade® stand out as the smallest and most comfortable face shield on the market.

Warning: Only use with front shield attached. Only use against liquids. Shield is not impact resistant. Do not use as protection against heat, open flames or sparks.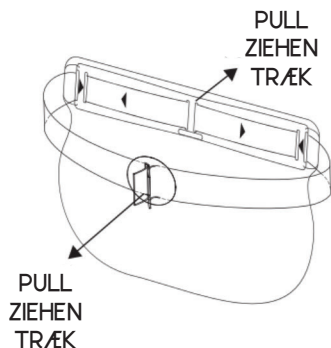

1

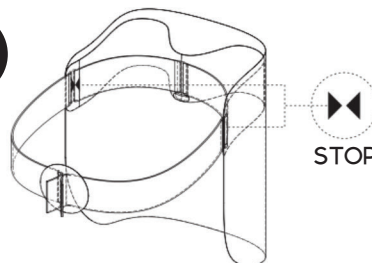US/GB 1: Use one hand to firmly hold the two ends of the elastic band. Hold with the other hand where the shield and the front of the elastic band meet in the middle. Then pull apart. Stop when arrowheads meet.

2: Place viraShade® on your head. Adjust the size by pulling or loosening the head adjustment in the back.

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